

INSTRUMAX® PINK

INSTRUMENT GRADE DISINFECTANT - LOW LEVEL

ARTG No: 135544

INSTRUMAX® PINK is a specially formulated instrument grade disinfectant-low level. INSTRUMAX® PINK helps reduce the risk of cross infection and is suitable for numerous applications.

INSTRUMAX® PINK is approved for use on environmental surfaces and non-critical medical devices including stethoscopes, thermometers, blood pressure cuffs, tourniquets, humidicribs and MRI machines. INSTRUMAX® PINK is also suitable for use on dentures, dental impressions and a wide range of non medical applications including training masks, mannequins used for CPR training and self contained breathing apparatus. Any applications where face masks are subjected to re-use and require on site disinfection may be suitable for INSTRUMAX® PINK.

BENEFITS:

- Suitable for numerous applications
- Kills a broad range of pathogenic bacteria
- Excellent materials compatibility
- Does not contain phenolics, chlorine or glutaraldehyde
- pH neutral
- Non-hazardous as per Safework Australia guidelines

The **INSTRUMAX® PINK** label lists that it is proven to kill:

- Acinetobacter
- Candida albicans
- Coronaviruses including SARS- CoV-2 (COVID-19)
- Enterococcus Faecalis (VRE)
- Escherichia coli (E coli)
- Hepatitis B Group virus
- Herpes Simplex virus
- Influenza virus
- Klebsiella pneumoniae (CPE/CRE)
- Proteus vulgaris
- Pseudomonas aeruginosa
- Salmonella choleraesuis
- Staphylococcus aureus (MRSA or Golden Staph)

**Independent test reports available on request.*

DIRECTIONS FOR USE:

1. Thoroughly pre-clean non-critical medical device in warm water with an instrument detergent (Matrix®, Medizyme, Sonidet® or Clinimax®) according to the equipment manufacturer's instructions. For environmental surfaces clean prior to disinfection with detergent (Mediclean or Speedy Clean).
2. **FOR DISINFECTION:** USE UNDILUTED. Apply sufficient INSTRUMAX® PINK to environmental surface or non-critical medical device to allow for a wet contact time of 10 minutes OR soak the parts directly in the INSTRUMAX® PINK solution for not less than 10 minutes.
3. Rinse off thoroughly after 10 minutes with clean running water and discard the used solution to the sink/sewer.

** 10-minute contact time is essential for thorough disinfection.*

4. Do not re-use the solution. The use of gloves is recommended whenever using disinfectants.

INSTRUMAX® PINK is not intended to be used on semi-critical or critical medical devices.

Note: INSTRUMAX® PINK can be used for both the cleaning and disinfection steps.

CHEMICAL COMPOSITION AND PROPERTIES:

Surfactant system	Blended
Colour	Light pink
pH	7.00 - 7.40
Solubility	Completely miscible with water
Specific gravity	0.99
Corrosion inhibitor	Not necessary
Foaming properties	Slight
Abrasives	Nil
Free caustic	Nil
Dangerous goods rating	Not applicable
Shelf life	18 months
Safety equipment	Gloves and eye protection are recommended

SAFETY DATA: Information is listed on the Safety Data Sheet.

PACK SIZES: 6x500mL trigger spray pack, 12x500mL squeeze pack, 2x5L carton.

INSTRUMAX® PINK has been tested for compatibility with a number of plastic materials with favourable results. However, due to the variety of plastic materials used in the construction of medical devices compatibility with any particular material cannot be automatically assumed. Overuse and prolonged soaking may lead to early deterioration of some plastic or rubber surfaces

SUITABLE USES:

ENVIRONMENTAL SURFACES AND NON-CRITICAL MEDICAL DEVICES. Consult the medical device manufacturer's instructions prior to use. INSTRUMAX® PINK will not damage non-critical medical devices with materials that include some plastics (such as nebuliser masks), glass, stainless steel or aluminium. Environmental surfaces or instruments other than medical devices should be treated in the same way as non-critical medical devices. INSTRUMAX® PINK is intended only for the disinfection of environmental surfaces and non-critical medical devices and should not be used for the disinfection of any critical or semi-critical medical device.

HEALTH AND SAFETY STATEMENT:

There is no way of predicting an individual's response to cleaning or disinfecting products. All cleaning and disinfecting products have a potential to cause skin and other irritation in susceptible individuals. Direct physical contact with the skin should be avoided. If irritation does occur when using this product or any other cleaning/disinfecting solution then the individual should be removed from direct contact with the solution until a satisfactory safe working environment for that individual is determined.

ENVIRONMENTAL STATEMENT:

Whiteley Corporation manufactures products that comply with Australian and International environmental policy. Whiteley Corporation aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley Corporation products are supplied in 100% recyclable containers. Whiteley Corporation and its staff are fully committed to ensuring that our Australian environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© This document remains property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.