

V-WIPES

HOSPITAL GRADE DISINFECTANT WIPES
AUST L 160901

ABOUT V-Wipes:

V-Wipes unique formula and high quality apertured fabric allow you to clean and disinfect in one simple step.

PRODUCT FEATURES:

- Hospital Grade Disinfectant Wipes.
- pH neutral – safe for use on most surfaces & equipment.
- Extremely durable and highly absorbent apertured fabric.
- Leaves surfaces streak free.
- Non hazardous.
- Does not contain phenolics or chlorine.
- Non flammable.
- Non corrosive to metals.

DIRECTIONS FOR USE:

Remove organic soiling from surface (blood, sweat, sputum etc) and clean surface using a V-Wipe. Dispose of soiled wipe. Disinfect clean surface using fresh V-Wipe. Ensure surface to be disinfected is contacted by clean and wet V-Wipe. Allow surface to dry.

This product is not intended to be used on Medical Devices. Do not mix with detergents other chemicals.

HELPS CONTROL INFECTION:

V-WIPES can be used for routine cleaning and can also be used for disinfecting where necessary. This makes for simplification of cleaning and disinfecting procedures.

EDUCATIONAL VIDEO AVAILABLE:

An Educational video is available on 'How to clean and disinfect blood spills using VIRACLEAN® and V-WIPES'. To watch the educational video visit www.whiteley.com.au/education

PRODUCT USE:

<i>Nursery</i>	Cleaning cribs, cots, change tables.
<i>CSSD</i>	Bench tops, stainless steel surfaces, sinks, trolleys, tap fittings.
<i>Toilet / Shower</i>	Sinks, tile surfaces, and all hard surfaces.
<i>Intensive Care Units</i>	Cleaning and disinfection of all hard surfaces.
<i>Manual Cleaning of Theatre Equipment</i>	V-WIPES can be used to clean and disinfect equipment and cables and will not adversely affect surfaces.
<i>Operating Theatres</i>	Cleaning and disinfecting of tables in theatre.
<i>Blood Spills</i>	V-WIPES are ideal for cleaning and disinfecting blood spills on all hard surfaces.
<i>Telephones</i>	V-WIPES are ideal for cleaning and disinfecting phones where large numbers of bacteria are found.
<i>Dental & GP Surgeries</i>	V-WIPES can be used in Dental & GP Surgeries for cleaning and disinfecting surfaces including chairs, stools, tables, benches and light fittings.
<i>Laboratories & Pathology Clinics</i>	V-WIPES are excellent for wiping down benches, disinfection of white rooms, cleaning and disinfection of fume cupboards, etc.

SAFETY DATA: Information is listed in the Safety Data Sheet.

PACK SIZE: 50 Wipes Flatpack (230x300mm)
100 Wipes Canister (180x250mm)

ENVIRONMENTAL STATEMENT:

Whiteley Corporation manufactures products that comply with Australian and International environmental policy. Whiteley Corporation aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley Corporation products are supplied in 100% recyclable containers. Whiteley Corporation and its staff are fully committed to ensuring that our Australian environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© This document remains property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.