

# MEDIRINSE

## RINSE AID

AUSTRALIAN MEDICAL DEVICE CLASS 1

ARTG NO. 125529

MEDIRINSE is a liquid rinse aid designed as a water dispersant for use in clinical washer/disinfectors and laboratory automatic dish/instrument washing machines. The primary function of MEDIRINSE is to disperse the rinse water to ensure the rapid drying of instruments and utensils.

**Guaranteed quality:** Use of only highly efficient ingredients ensures consistent quality performance, free from rejects. MEDIRINSE produces fast spot and streak free drying of all washed articles.

**Consistent destaining:** Excellent rinsibility gives the brightest possible results with less water spotting and streaks after heavy loads.

**Better machine performance:** MEDIRINSE has an in-built protection mechanism which assures continuous trouble-free machine operation with clean jets and less pump troubles.

### BENEFITS:

- Rapid Water Dispersal
- Ensures faster drying of instruments
- Concentrated formula

### DIRECTIONS FOR USE:

MEDIRINSE should be used after the machine detergent (Medigleam, Sonex, Genesis or Polaris) has been through the cleaning cycle in a washer/disinfector or automatic washing machine. MEDIRINSE should be dispensed through an automated system at 0.5 – 1.0 mL/L of water to minimise instrument or load drying times.

### CHEMICAL COMPOSITION AND PROPERTIES:

Surfactant System: ..... Non-ionic

Colour [Concentrated]: ..... Blue Liquid

Colour [Dilute solutions]: ..... Colourless

Odour: ..... Odourless

pH [Concentrate]: ..... 7.0-8.0

Solubility: ..... Completely Soluble

Rinsibility: ..... Excellent, residue free

Specific Gravity: ..... 1.006

Foam Properties: ..... Low foaming

Dangerous Goods Rating: ..... None

Shelf Life: ..... 2 years

**MATERIALS COMPATIBILITY**

MEDIRINSE has been used on a vast range of materials over several decades and has excellent general materials compatibility. However it has not been tested on every type of plastic or material. Should you notice degradation, cracking or a change in the colour of materials please immediately discontinue use of MEDIRINSE on that material.

**ABOUT LMS:**

Our comprehensive learning platform offers a broad range of information including the latest information on Infection Prevention and Control, recommended products and cleaning processes for individual markets.



Scan to register for Whiteley's  
Online Learning Centre



**SAFETY DATA:** Information is listed in the Safety Data Sheet.

**PACK SIZE:** 2 x 5 Litre  
2 x 5 Litre Wide Neck,  
15 Litre

**HEALTH AND SAFETY STATEMENT:**

There is no way of predicting an individual's response to cleaning or disinfecting products. All cleaning and disinfecting products have potential to cause skin and other irritation in susceptible individuals. Direct physical contact with the skin should be avoided. If irritation does occur when using this product or any other cleaning/ disinfecting solution then the individual should be removed from direct contact with the solution until a satisfactory safe working environment for that individual is determined.

**ENVIRONMENTAL STATEMENT:**

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley products are supplied in 100% recyclable containers. Whiteley and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© 2023 This document remains the property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.