

Devac

For ultra clean suction lines


HOW TO USE DEVAC TWO PART SYSTEM

Devac DAILY


Devac DAILY is highly effective in removing debris and bio-burden from evacuation lines and pumps. It provides excellent maintenance and ensures optimal suction on all dental aspiration systems. The specially formulated alkaline based formulation is non-foaming.

Devac WEEKLY

Devac WEEKLY is designed to provide heavy duty removal of debris and bio-burden from evacuation lines and pumps. Used regularly it provides excellent maintenance and ensures optimal suction on all dental aspiration systems. The specially formulated acid-based formulation is non-foaming.

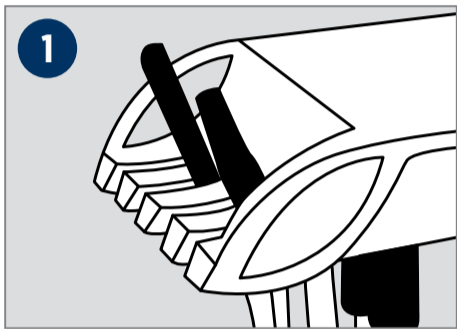


1 At the beginning of the day and between patients flush lines thoroughly with water.

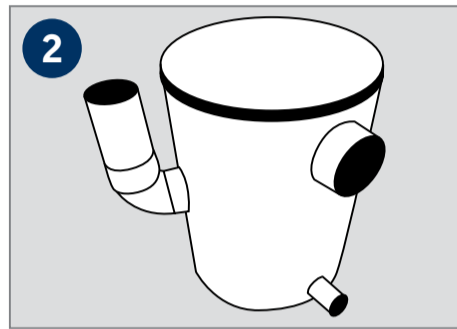

2 Check and clean all traps




3 At the end of the day, use **Devac DAILY** to remove bioburden and soft deposits.

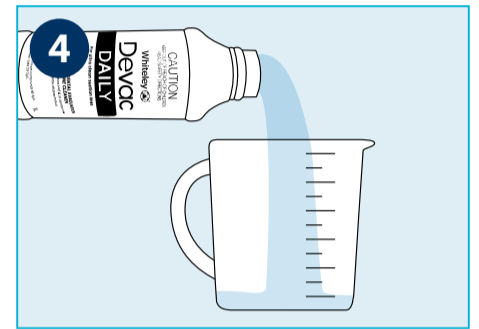


4 Add 25mL of **Devac DAILY** to jug.



5 Add water to make up to 1000ml [1L], then mix well by stirring.

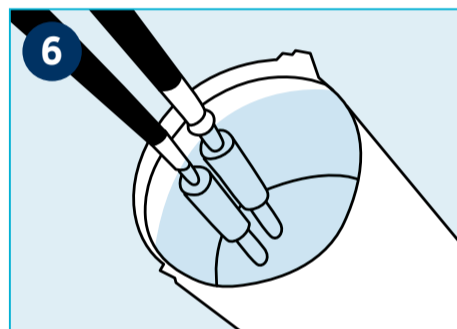

6 Fill evacuation line with the solution.

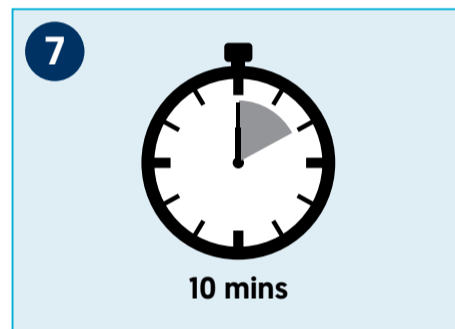

7 Allow to stand for a minimum of 10 minutes.




8 If necessary, also clean the spittoon bowl with the solution.



9 At the end of the 10 minute standing time, flush with 1L of water.




10 On the last day of operations for the week, use **Devac WEEKLY** in replacement of **Devac DAILY**, to purge lines and help remove hard deposits.

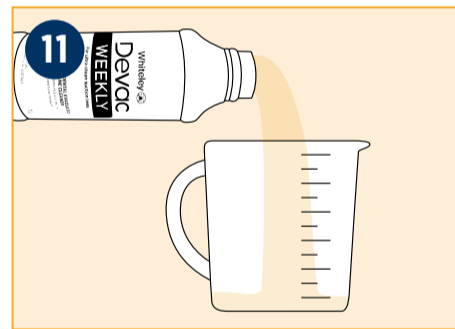


11 Add 25mL of **Devac WEEKLY** to jug.



12 Add water to make up to 1000ml [1L], then mix well by stirring.

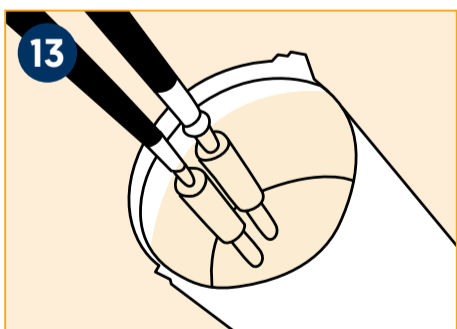

13 Fill evacuation line with the solution.

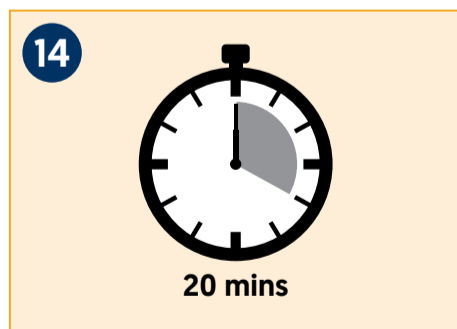

14 Allow to stand for a minimum of 20 minutes.



15 At the end of the 20 minute standing time, flush with 1L of water.

ADDITIONAL CONSIDERATIONS

- Surgery/heavy blood and tissue procedures should be purged with evacuation solution after patient.
- Re-check all traps and clean at the end of the day.
- Replace filters and traps when required (check your equipment maintenance manual).

CAUTION: Only use Devac on suction lines. DO NOT use on water lines or mix **Devac DAILY** and **Devac WEEKLY** solutions together.

For further information contact your Whiteley Representative or call our Product Support hotline on **1800 833 566**

	PACKSIZE	CODE
Devac DAILY	6 x 1L	040232
Devac WEEKLY	6 x 1L	040235