

Devac

For ultra clean suction lines

Whiteley 

HOW TO USE DEVAC TWO PART SYSTEM

Devac DAILY

Devac DAILY is highly effective in removing debris and bio-burden from evacuation lines and pumps. It provides excellent maintenance and ensures optimal suction on all dental aspiration systems. The specially formulated alkaline based formulation is non-foaming.

Devac WEEKLY

Devac WEEKLY is designed to provide heavy duty removal of debris and bio-burden from evacuation lines and pumps. Used regularly it provides excellent maintenance and ensures optimal suction on all dental aspiration systems. The specially formulated acid-based formulation is non-foaming.



1

At the beginning of the day and between patients flush lines thoroughly with water.



2

Remove and clean all traps of the suction line system. Then insert these back into place once cleaned.



3

At the end of the day, use **Devac DAILY** to remove bio-burden and soft deposits.



4

Measure 25mL of **Devac DAILY**.



5

Pour 25mL of **Devac DAILY** into a 1L jug. Then fill jug with water, stopping at the 1L mark.



6

Place suction lines into the **Devac DAILY** solution and aspirate 900mL through the suction lines.



7

Leave the suction lines to stand for 10 minutes.



8

Pour the remaining 100mL of **Devac DAILY** solution into the spittoon.



9

After 10 minutes, aspirate the suction lines with 900mL of water and pour the remaining 100mL of water into the spittoon.



10

On the last day of operations for the week, use **Devac WEEKLY** in replacement of **Devac DAILY**, to purge lines and help remove hard deposits.



11

Measure 25mL of **Devac WEEKLY**.



12

Pour 25mL of **Devac WEEKLY** into a 1L jug. Then fill jug with water, stopping at the 1L mark.



13

Place suction lines into the **Devac WEEKLY** solution and aspirate 900mL through the suction lines.



14

Leave the suction lines to stand for 20 minutes.



15

Pour the remaining 100mL of **Devac WEEKLY** solution into the spittoon.



16

After 20 minutes, aspirate the suction lines with 900mL of water and pour the remaining 100mL of water into the spittoon.

ADDITIONAL CONSIDERATIONS

- Surgery/heavy blood and tissue procedures should be purged with evacuation solution after patient.
- Re-check all traps and clean at the end of the day.
- Replace filters and traps when required (check your equipment maintenance manual).

CAUTION: Only use Devac on suction lines. **DO NOT** use on water lines or mix **Devac DAILY** and **Devac WEEKLY** solutions together.

PACKSIZE CODE

| | | |
|--------------|--------|--------|
| Devac DAILY | 6 x 1L | 040232 |
| Devac WEEKLY | 6 x 1L | 040235 |

ABN No. 24 000 906 678
PO Box 1076, North Sydney NSW 2059
Product Support Hotline: 1800 833 566
www.whiteley.com.au

NZBN: 9429041224243
PO Box 22-519, Otahuhu, Auckland 1640
Product Support Hotline: 0800 257 352
www.whiteley.co.nz

Whiteley 

Protecting lives and livelihoods since 1933