

MEDI-GLEAM

HEAVY DUTY MACHINE LIQUID DETERGENT

AUSTRALIAN MEDICAL DEVICE CLASS 1

ARTG NO. 125528

MEDI-GLEAM is a concentrated alkaline, non-foaming liquid detergent intended for heavy-duty mechanical washing. It can be used in hospital washer/disinfectors, utensil and pan washers and is highly efficient in removing biological soils found in healthcare, clinics and laboratories. It is designed to remove biological soils on medical instruments, anaesthetic equipment, stainless steel, plastics and glassware. Highly buffered alkali reserve, good wetting and excellent rinsability all contribute to quick, sure, regular cleaning. MEDI-GLEAM contains a high level of sequestrant, which ensures excellent performance in hard water areas and reduces the possibility of scale formation within the machine.

- Guaranteed quality cleaning:** Use of only highly efficient ingredients ensures consistent quality cleaning, free from rejects.
- Consistent destaining:** High buffered alkali level ensures sustained cleaning and destaining. Excellent rinsability gives the brightest possible results with less water spotting and streaks after heavy loads.
- Better machine performance:** In-built protection mechanism assures continuous trouble free machine operation with clean jets, less pump trouble and freedom from scale in machines.
- Excellent rinsability:** Very little compound is needed for excellent results. Combination of better detergency, removal of ingrained surface films,

BENEFITS:

- In-built scale inhibitor
- Ideal for use in hard or soft water
- Phosphate free
- Concentrated formula

DIRECTIONS FOR USE:

Use 2-5 mL per litre of water. Concentration required will vary depending upon type of machine load, water hardness and dilution in the cleaning solution tank during normal washing cycles. Automated systems provide the safest means of chemical use in clinical departments, therefore greatly reducing assessable risk in your department. Should particular machines require a rinsing compound MEDIRINSE or ARID should be used. MEDIRINSE or ARID is fed neat into rinse injectors where fitted to washer/disinfectors.

CHEMICAL COMPOSITION AND PROPERTIES:

Colour (concentrate):..... Colourless - slightly yellow
 Colour (dilute solutions): Colourless - very pale yellow
 Odour: Odourless
 Solubility: Completely Soluble
 Rinsability: Excellent, no residues
 Specific Gravity: 1.210

% Alkalinity: 13.0-14.0
 Phosphates: Nil
 Toxicity: Non toxic
 Dangerous Goods Rating: Class 8 [corrosive]
 Shelf Life: 2 years

MATERIALS COMPATIBILITY

MEDI-GLEAM has been used on a vast range of materials over several decades and has good general materials compatibility. However, do not use on anodised aluminium [such as sterilising trays], or other sensitive alloys. MEDI-GLEAM has not been tested on every type of plastic or material. Should you notice degradation, cracking or a change in the colour of materials please immediately discontinue use of MEDI-GLEAM on that material.

ABOUT LMS:

Our comprehensive learning platform offers a broad range of information including the latest information on Infection Prevention and Control, recommended products and cleaning processes for individual markets.



Scan to register for Whiteley's
Online Learning Centre



SAFETY DATA: Information is listed in the Safety Data Sheet.

PACK SIZE: 2 x 5 Litre
 2x5L Squat
 2 x 5 Litre Wide Neck,
 15 Litre

HEALTH AND SAFETY STATEMENT:

There is no way of predicting an individual's response to cleaning or disinfecting products. All cleaning and disinfecting products have potential to cause skin and other irritation in susceptible individuals. Direct physical contact with the skin should be avoided. If irritation does occur when using this product or any other cleaning/ disinfecting solution then the individual should be removed from direct contact with the solution until a satisfactory safe working environment for that individual is determined.

ENVIRONMENTAL STATEMENT:

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley products are supplied in 100% recyclable containers. Whiteley and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© 2024 This document remains the property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.