

MEDICLEAN

READY TO USE HOSPITAL GRADE NEUTRAL DETERGENT

ARTG NO. 125529

MEDICLEAN is a ready to use neutral detergent that gives a streak free finish on all surfaces. MEDICLEAN uses Whiteley's emulsion systems technology which enhances MEDICLEAN's ability to remove oils and dirt that are encountered in everyday cleaning.

MEDICLEAN is a class 1 medical device and can be used to clean other medical device surfaces. MEDICLEAN has been formulated to replace most cleaning products and is totally compatible for cleaning before disinfection with Viraclean, Surfex® and other surface disinfectants.

MEDICLEAN aids in the infection control process because of its single use packaging which eliminates the need for decanting or dispensing of detergents. The elimination of detergent dispensing systems means a reduction of risk associated with bacterial growth and contamination of dispensing systems and refillable bottles. Coupled with Viraclean® this makes for simplification of the cleaning and disinfection process.

MEDICLEAN packaging is 100% recyclable which means it has a low environmental impact.

Do not mix with chemicals.

BENEFITS:

- Provides a streak free finish
- Neutral pH
- Non corrosive, non toxic
- Improves Infection Control by avoiding contamination through re-used containers
- Packaging 100% recyclable
- Class 1 Medical Device

MEDICLEAN can be used on all surfaces including the following:

- Medical devices
- Nursery surfaces & equipment
- CSSD
- Intensive Care Units
- Theatre Equipment
- Ward areas
- Telephones

DIRECTIONS FOR USE:

Refer to label prior to first use. Apply onto surfaces and wipe off for a streak-free finish.

CHEMICAL COMPOSITION AND PROPERTIES:

Surfactant System: Blended

Colour: Pale pink

pH: Neutral

Solubility: 100% in water
 Specific Gravity: 1.00
 Abrasives: Nil
 Free Caustics: Nil
 Toxicity: Non toxic
 Dangerous Goods Rating: Nil
 Shelf Life: 15 months
 Safety Equipment: Gloves recommended

MATERIALS COMPATIBILITY

MEDICLEAN has been used on a vast range of materials over several decades and has excellent general materials compatibility. However it has not been tested on every type of plastic or material. Should you notice degradation, cracking or a change in the colour of materials please immediately discontinue use of MEDICLEAN on that material.

ABOUT LMS:

Our comprehensive learning platform offers a broad range of information including the latest information on Infection Prevention and Control, recommended products and cleaning processes for individual markets.



Scan to register for Whiteley's
Online Learning Centre



SAFETY DATA: Information is listed in the Safety Data Sheet.

PACK SIZE: 12 x 500mL

ENVIRONMENTAL STATEMENT:

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley products are supplied in 100% recyclable containers. Whiteley and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© 2024 This document remains the property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.